


Brief instructions for use

Server RT-985, Server Vis-a-Vis

VÖLKER

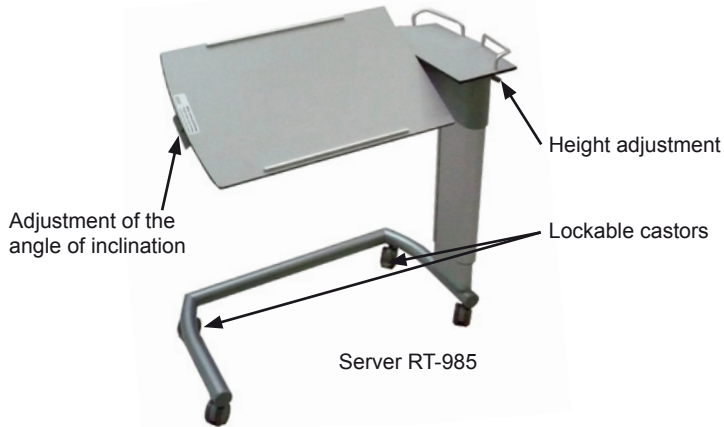

Height adjustment

To bring the tray upwards, pull it up slowly. To bring it downwards, push the handle upwards and let the tray down carefully.



Adjusting the angle of inclination

To adjust the angle of inclination of the tray, push the handle upwards and tip the tray in the desired direction.



CAUTION Do not lean on the server!



CAUTION The server is not suitable for transporting persons!



CAUTION Secure the castors of the server to make sure that it cannot roll away!



CAUTION There must be no objects on the tray while you adjust the angle of inclination of the tray!

NOTE The total weight of the objects on the server may not exceed 15 kg!