


# Brief instructions for use

## K 941, K 941/2, P 941, P 945 bedside tables

VÖLKER

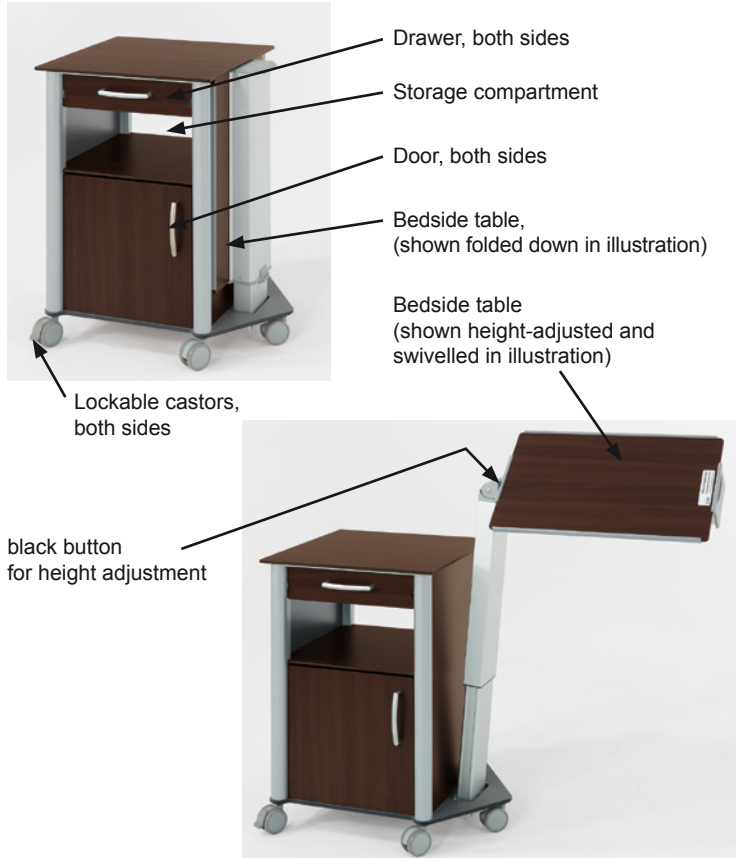

### Folding out the bedside table

To fold out the bedside table, push it slightly downwards. Then pull it side-ways away from the bedside table and raise it slowly until it stops. Then fold the bedside table into the horizontal position.

### Adjusting the angle of inclination

To adjust the bedside table's angle of inclination, lift it slightly and swivel it in the required direction. Then lower the bedside table again until it engages.

### Height adjustment

To lower the bedside table, press the black height adjustment button and push the bedside table slowly downwards as you do so. The button does not need to be pressed to adjust the table upwards.

### Folding the bedside table in

To fold the bedside table in, swivel it if necessary to the horizontal position and then fold it upwards. Now slowly push it downwards and swivel it under the bedside table top.

**NOTE** The total weight of the objects on the bedside table must not exceed 10 kg!



**CAUTION** Do not lean on the bedside table!



**CAUTION** The bedside table is not suitable for transporting people!!



**CAUTION** Secure the bedside table's castors to ensure it does not roll away!



**CAUTION** There must be no objects on the bedside table while you are adjusting its angle of inclination!