


Installation guide for model ZK-992 2.0/2.1/2.2

Clip-on 80 mm side rail extension

VÖLKER

Accessories

Item	Article	Qty.
1	Side rail extension backrest section	2
2	Side rail extension lower leg section	2

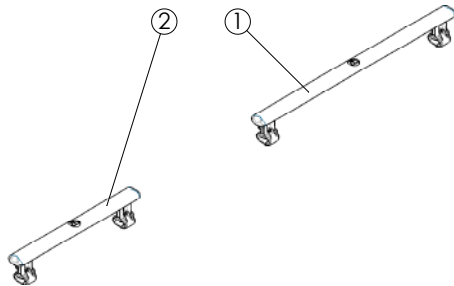

Pre-assembly

The individual side rail extensions are already fully pre-assembled.

Installation

1. Unscrew the two knurled screws on each side rail extension and open the clips.
2. Push the clips from the inside of the bed onto the butt edges of the raised side rails and align the side rail extension in the centre (Fig. 2).
3. Close the clips by screwing the knurled screws back into the sleeve nuts (Fig. 3). The knurled screws must be aligned with the outside edge of the bed.
4. Check the connections to ensure they are tight (no Fig.).

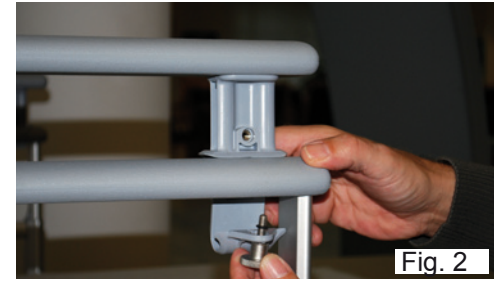


Fig. 2



Fig. 3

