

# Assembly instructions Restraint holders

## Models ZP-3051/3 and ZK-972/3


for care beds built after 05/2004 and hospital beds built after 11/2001

# VÖLKER



**The lying surfaces must never be adjusted while the patient is restrained and must always be in the lowest position.**


**The lying surface adjustment functions must be locked when a patient is restrained, and the hand control must be kept out of the occupant's/patient's reach!**

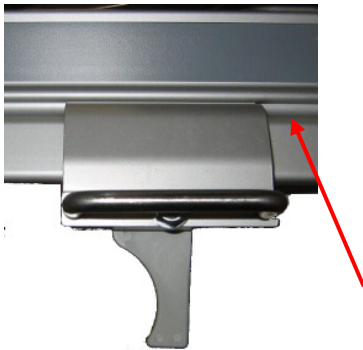

1. Clip the restraint holder into the side part of the care or hospital bed. Before doing so, make sure that the screw connection of the lever is fastened tightly, i.e. the lever should require firm pressure to move it.



2. Press the restraint holder onto the side part and press the lever upwards to the end stop.
3. If the lever needs to be secured against accidental loosening, the union nut must be tightened with a 13 wrench.

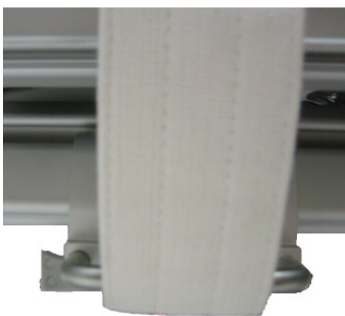

4. Now fasten the restraint system to the restraint holders in accordance with the manufacturers' instructions.



- Restraint systems such as belts or straps should only be used exactly as specified by the manufacturer.
- The restraint holders can be positioned according to requirements, depending on where the restraint system is to be used. Please always follow the manufacturer's instructions here.
- If restraint systems in the form of abdominal belts are used, then it must be ensured that the side rails are completely raised. In beds with split side rails, the gap in the middle of the rails must always be closed using a rail spacer!

Völker GmbH • Wullener Feld 79 • 58454 Witten - Germany  
Telephone +49 2302 960 96-0 • Fax +49 2302 960 96-16  
[www.voelker.de](http://www.voelker.de) • [service@voelker.de](mailto:service@voelker.de)