



A. Slide rails

To raise the side rails, lift the side rail by gripping the upper rail indent and lifting until the rail is at the uppermost position. Raise the head end of the side rail and then the foot end.

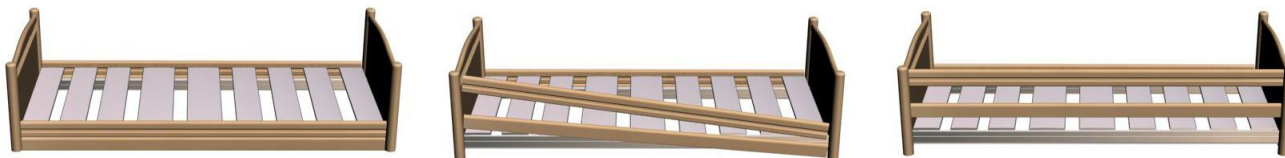

To lower the side rails, lower the side rail by gripping the upper rail indent, raising the rail slightly, push the side rail release button, and then lower the side rail. Lower the foot end of the side rail and then lower the head end.

B. Hand control unit

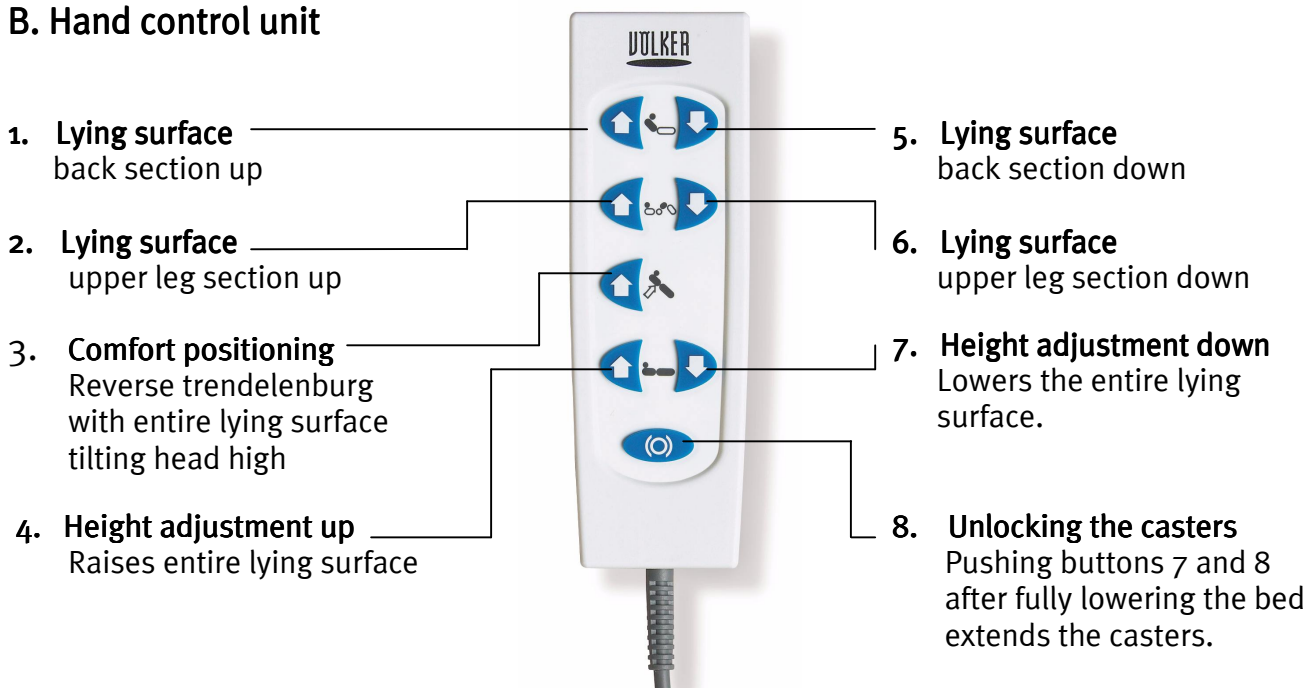

C. Carergiver's console

1. Height adjustment: „UP/DOWN“ :

„Double-click-function up/down“ allows continuous **up** or **down** movement of the lying surface with an instant stop by pushing any button.

2. Hand control lock

Locks all hand control and console unit functions - except trendelenburg.

3. Trendelenburg position

tilts the entire lying surface with the foot section high.

4. Reverse Trendelenburg position

Tilts the entire lying surface with the head section high.

5. lying surface : back section


Raises and lowers the back section of the lying surface

6. lying surface : leg section


Raises and lowers the leg section of the lying surface

9. Caster Button



Pushing the caster button at the same time as the height adjustment down button – extends the casters and allows the bed to be moved.

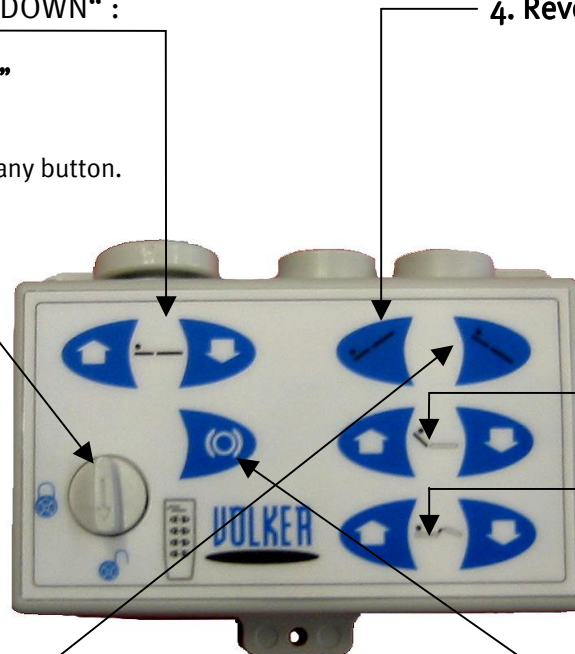

D. Raised foot section adjustment

Pushing button 2  moves the bed to the „bend knee“-position.

The foot section can then be raised manually.

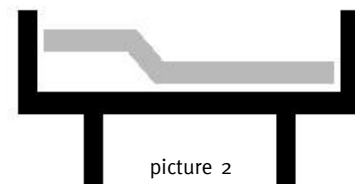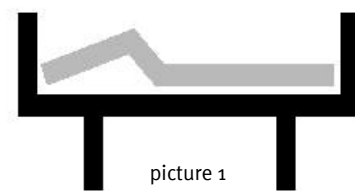

Review the complete operation manual before operating the bed.