


Quick reference guide for healthcare bed model 2080


A. Side rails

To raise the side rails, lift the side rail by gripping the upper rail indent and lifting until the rail is at the uppermost position. Raise the head end of the side rail and then the foot end.

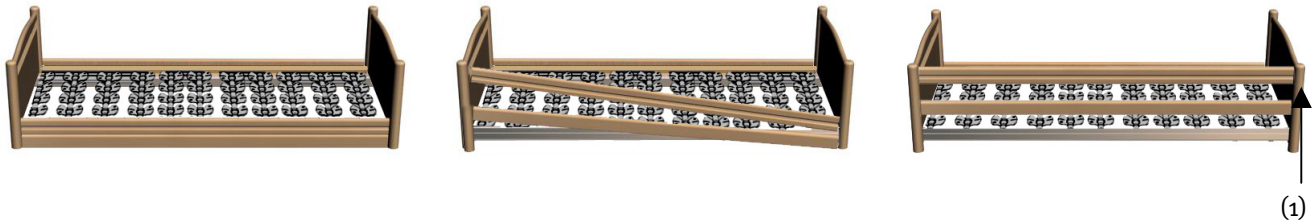

To lower the side rails, lower the side rail by gripping the upper rail indent, raising the rail slightly, push the side rail release button (1), and then lower the side rail. Lower the foot end of the side rail and then lower the head end.

B. Manual control unit

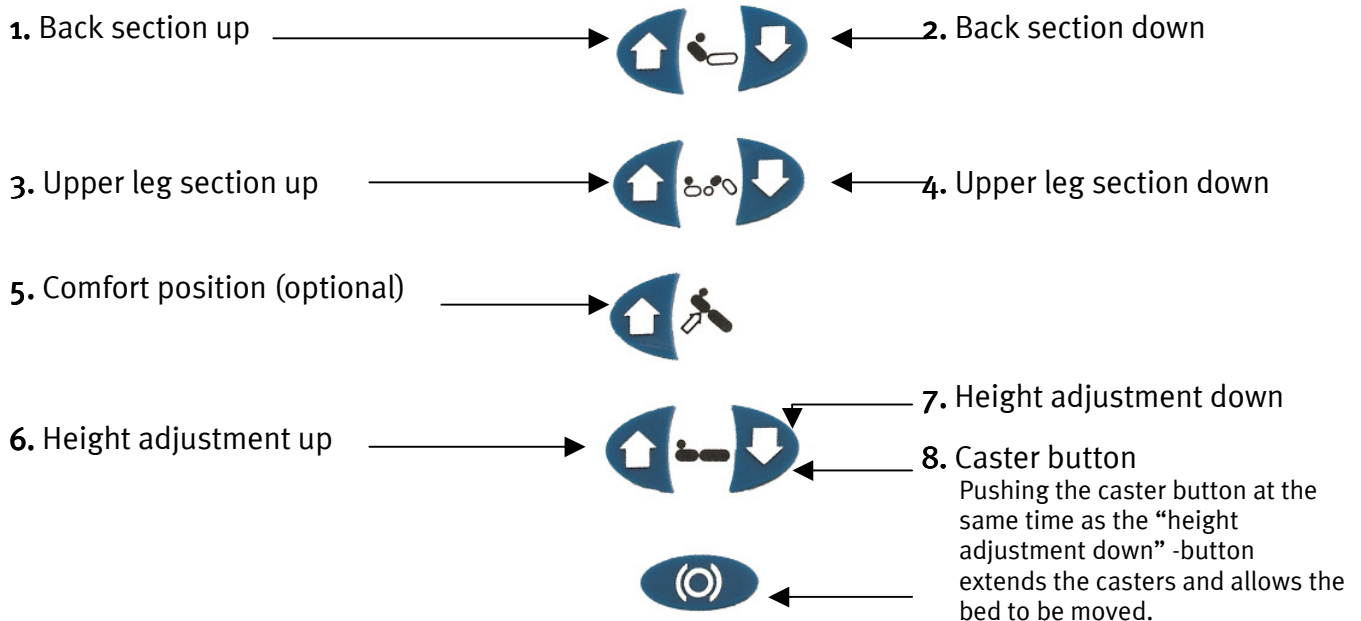

Quick reference guide for healthcare bed model 2080

C. Caregiver`s console (optional)

Trendelenburg


Tilt's the entire lying surface with the foot section high.

Comfort position

Tilt's the entire lying surface with the head section high.

Back section

Raises and lowers the back section.

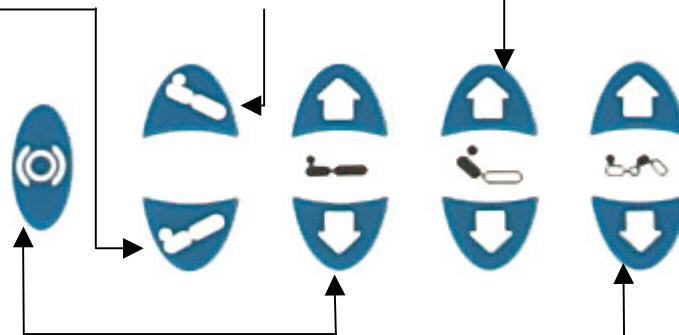

Caster button

Pushing the caster button at the same time as the "height adjustment down" -button extends the casters and allows the bed to be moved.

Height adjustment *

Raises and lowers the lying surface of the bed

Leg section

Raises and lowers the leg section.

D. Control lock

a) Manual control unit

By activating the control lock on the backside of the manual control unit, you lock all functions of the manual control unit.

b) Caregiver`s concole

By activating the control lock on the caregiver`s console, you lock all functions of the caregiver`s console and the manual control unit.



Activate the control lock



Deactivate the control lock



If the functions of the bed can not be operated, please check the control lock on the manual control unit and the caregiver`s console.



Please pay attention to the information in the operating instructions.