

## Quick reference guide for healthcare bed model 3080


### A. Side rails

#### To raise a side rail section,

pull the section horizontally out of its location to end-stop and fold it upwards. In order to adjust the height of the side rail, pull the telescopic part up to the stop.

#### To lower a side rail section completely,

1. Press the two buttons on the inner side of the frame, right above the cross-member, so that the height-adjustable side rail section can be put into its lowest position.
2. Operate the (2a) release button labelled “Push” at the bottom of the side rail part and (2b) tilt it horizontally to the side so that it runs parallel to the floor.
3. Push the side rail completely under the lying surface.

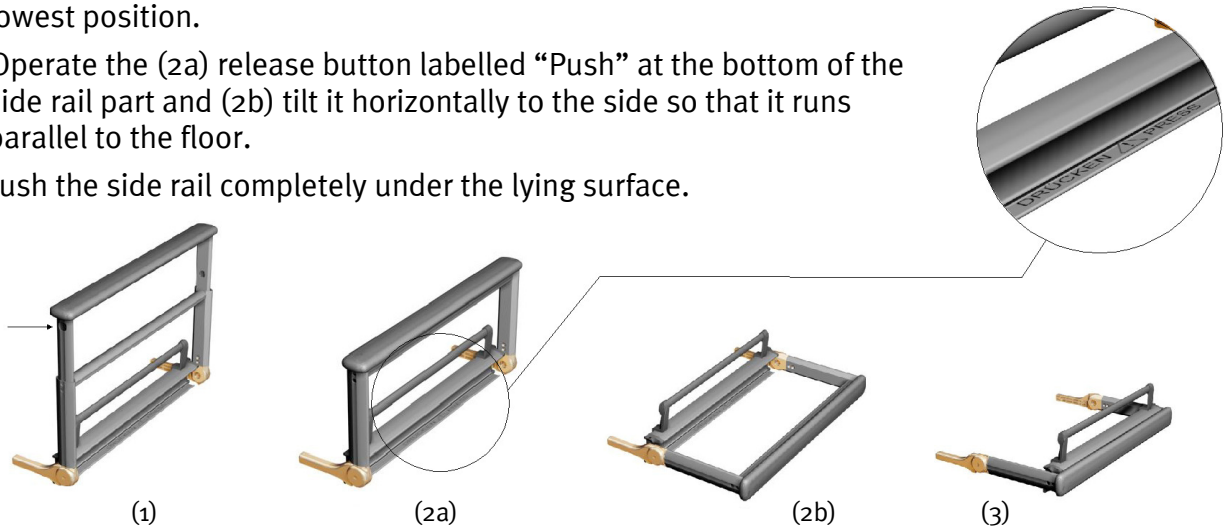

### B. Manual control unit

1. Back section up → ← 2. Back section down
3. Upper leg section up → ← 4. Upper leg section down
5. Comfort position (optional) →
6. Height adjustment up → ← 7. Height adjustment down
8. Caster button  
Pushing the caster button at the same time as the “height adjustment down” -button extends the casters and allows the bed to be moved.

## Quick reference guide for healthcare bed model 3080

### C. Caregiver`s console (optional)

#### Trendelenburg


Tilt's the entire lying surface with the foot section high.

#### Comfort position

Tilt's the entire lying surface with the head section high.

#### Back section

Raises and lowers the back section.

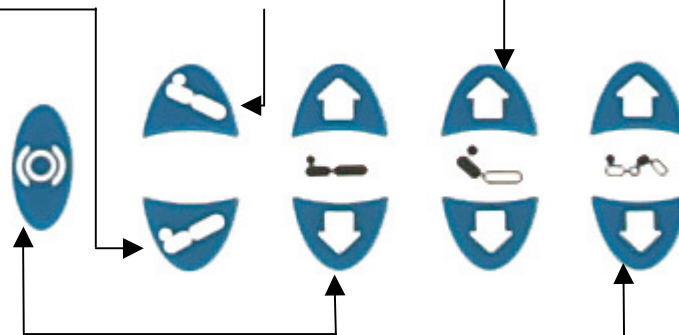

#### Caster button

Pushing the caster button at the same time as the "height adjustment down" -button extends the casters and allows the bed to be moved.

#### Height adjustment \*

Raises and lowers the lying surface of the bed

#### Leg section

Raises and lowers the leg section.

### D. Control lock

#### a) Manual control unit

By activating the control lock on the backside of the manual control unit, you lock all functions of the manual control unit.

#### b) Caregiver`s concole

By activating the control lock on the caregiver`s console, you lock all functions of the caregiver`s console and the manual control unit.



Activate the control lock



Deactivate the control lock



If the functions of the bed can not be operated, please check the control lock on the manual control unit and the caregiver`s console.



Please pay attention to the information in the operating instructions.