


# Use of assist rail spacer

# VOELKER



pic. 1

ZK-969  
ZK-969/2  
ZK-1069  
ZP-3067  
ZP-3067/2

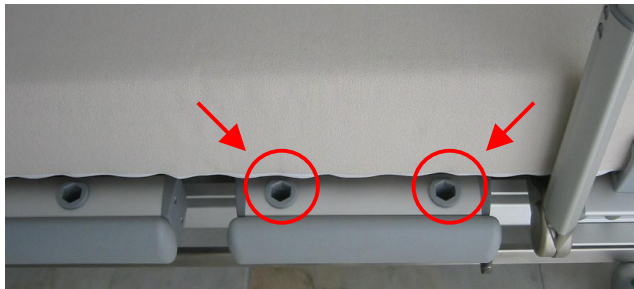

pic. 2

To provide a fully-length sideprotection, insert the assist rail spacer into the intended sleeves at the bed. The spacer must be placed at the outer side of the siderails!



pic. 3

Check if the security lug is properly fastened by pulling the assist rail spacer to the top. If this is possible, push the assist rail spacer back to the bottom and push the lug bedwards (picture 4). Now it should not be possible to pull out the assist rail spacer.



pic. 4

To remove the assist rail spacer, move the security lug little towards your position, so that it fits through the sleeve. The spacer can be pulled out now.



- Make sure that the assist rails – or parts of it – are completely raised and locked.
- To avoid injuries from clamping, it has to be ensured that no person gets in contact with the sideprotection when using the electrical functions.
- We recommend to lock all functions of the hand control.
- The assist rail spacer can only be used with our healthcare bed 3080 or hospital beds model S960.