

### Changes to the models 5380, 5380 K as of manufacture date 19.11.2012:

The safe working load for the models 5380 and 5380 K has been increased to 200 kg as of manufacture date 19.11.2012.

As a result, the maximum occupant / patient weights, specified in the chapter "Intended Use" of the instructions for use for the models 5380, 5380 K, are also increasing according to the following table:

Model	Safe working load	Max. occupant/patient weight	
		in application environment 1,2	in application environment 3, 4, 5
Beds as of manufacture date 19.11.2012:			
5380	200 kg	135 kg	165 kg
5380 K	200 kg	135 kg	165 kg
Beds before manufacture date 19.11.2012:			
5380	170 kg	105 kg	135 kg
5380 K	170 kg	105 kg	135 kg

### Changes to the models 5380, 5380 K as of manufacture month April 2013 or to converted beds before manufacture month April 2013:

Beds as of manufacture month April 2013 or converted beds before manufacture month April 2013 are equipped with a battery compartment for 4 batteries (as opposed to the previous compartment for just one battery).

Please use batteries according to the following specifications:  
 9 V block battery(-ies)  
 (alkali manganese primary cell, commercially available),  
 1 or 4 depending on the battery compartment

