


# Brief instruction for using rail infill panels

ZK-969/3, ZP-3067/2, ZK-969/2, ZP-3067, ZK-969, ZP-1069 for beds from year of construction mid 2003

**VÖLKER**

A **Hilti-Rom** Company

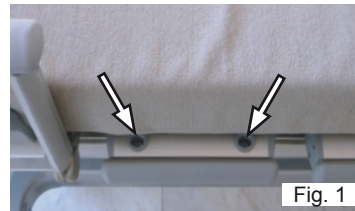

The rail infill panels close the middle gap between the side rails on Völker beds so as to enable a continuous side rail solution.

## Raising the side rails

1. First place the side rails on the bed. Follow the notes in the instructions for use or brief instructions for this.

## Mounting the rail infill panel

1. Guide the pins of the rail infill panel into the insertion sleeves in the center of the bed frame (Fig. 1). Make sure that the rail infill panel covers the side rails on the outside (Fig. 2).
2. Check whether the pendulum locking is completely latched into place by attempting to pull out the rail infill panel upwards. If you succeed in doing this, press the rail infill panel fully down again and adjust the pendulum until the rail infill panel can no longer be pulled out (Fig. 3).



## Removing the rail infill panel

1. To remove the rail infill panel from the holder, move the pendulum into a vertical position and pull the rail infill panel out upwards at the same time (Fig. 3).



**WARNING** Make sure that the side rails or parts of these are fully set up and latched into place. When activating the electric lying surface adjustment or side rails, it must be ensured that the patient/occupant has neither contact with the side rails, nor any part of the body protrudes through the side rails. We expressly recommend that you lock the functions of the hand control.

**NOTE** Please observe the notes in the bed's detailed instructions for use!